

Covid-19 Antigen Test Information

This is a SELF-ADMINISTERED antigen test. If you have any questions along the way we are here to help. Please follow the steps below.

1. Sign in (~5 min)

- a. Download the NAVICA™ app. Scan the QR code below to download the app.
- b. Please create an account.
 - c. If you have already completed this step, please go to step 1e.
- d. Once the account set up has been completed the app will say “Welcome to NAVICA™” press continue at the bottom of that page.
- e. Next please press “I am at a Test Site” and drive to station 2.





2. Testing (~5 min)

- a. Please have any form of ID on the dashboard of your car. The tester will double check your ID to the NAVICA™ account.
- b. Press “Display NAVICA™ ID” on the app. The tester will scan your unique QR code.
- c. The tester will scan your test card into the administration app and prepare the card for your nasal sample.
- d. The prepared card and nasal swab will be handed to you.

3. Sample Collection (~5 min)

- a. Carefully insert the swab into the nostril exhibiting the most visible drainage or the nostril with the most congestion.
- b. Gently rotate and push the swab up until resistance is met. This should be less than one inch into the nostril.
- c. Firmly rotate the swab in a circular path against the nasal wall 5 times.
- d. Using the same swab repeat this procedure in the other nostril.

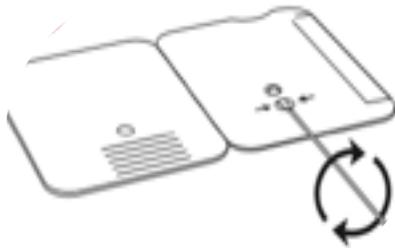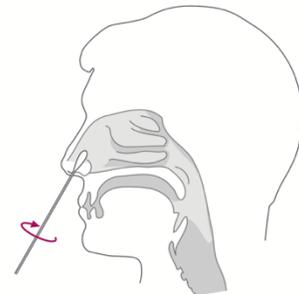

- e. Insert the nasal swab into the prepared test card through the **BOTTOM** hole first and push it upwards so the swab is visible through the top hole.
- f. Rotate the swab clockwise (to the right) three times. **DO NOT REMOVE THE SWAB**
- g. Peel off the adhesive liner on the right of the card. Close and securely seal the card.
- h. Set a timer for 15 minutes and leave the card in an undisturbed location. Continue driving with the flow of traffic up to station 3.

4. Sample Results:
- At the fifteen-minute mark a member of our team will review your results a station 3. If it has been longer than 20 minutes, please flag a staff member down.
 - When a staff member approaches your car, please have your sealed card ready to be scanned. Just hold it up for them and they will track your results on their iPad.
 - Please drop you test card and swab into the red biohazard trash bin. Please do not throw other trash in this bin.
 - You will be notified of your results by a notification on your NAVICA app.
 - You will be given educational material based on your test results.

Map of test site

